# BIOMET®

## MICROFIXATION, INC.

**1520 Tradeport Drive**  
**Jacksonville, Florida 32218-2480**  
**904-741-4400 • FAX 904-741-4500**

**01-50-1049**  
**Revision: C**  
**Date: 3/16/07**

### WARNINGS AND PRECAUTIONS FOR PECTUS SUPPORT BAR SYSTEM

### ATTENTION OPERATING SURGEON

#### DESCRIPTION

The Biomet Microfixation Pectus Support Bar and stabilizers are surgical implants intended to aid treatment of Pectus Excavatum deformity. The Pectus Support Bar provides the surgeon with a means to reposition bony structures (sternum, breastbone) by applying gradual force outwardly eliminating the funnel shape deformity. The device should be removed when remodeling is evident. The Pectus Support Bar and stabilizers are made from Stainless Steel, ASTM F 138.

#### INDICATION

Pectus Excavatum and other sternal deformities.

#### CONTRAINDICATIONS

- Patients with mental or neurological conditions who are unwilling or incapable of following instructions.
- Patients presenting metal sensitivity reactions.
- Patients with insufficient quantity or quality of bone or fibrous tissue to allow remodeling.
- Infection

#### WARNINGS AND PRECAUTIONS

The Pectus Support Bar provides the surgeon with a means of treating Pectus Excavatum, funnel chest, a congenital deformity often accompanied by shortness of breath in children. The device is not intended to replace chest wall structures. While the device is intended to expand the chest wall cavity eliminating the features of the deformity, the degree of initial reshaping and permanent remodeling for each case cannot be predetermined.

The surgeon is to be thoroughly familiar with the implants and the surgical procedure prior to surgery. The correct selection and placement of the implant is important. Preoperative planning to determine the most appropriate size and final position of the implant is required. The surgeon should always clearly identify, resect and remove the device at a hole. The implant can become dislodged, shift, or flip as a result of improper device selection, improper stabilization, not suturing the device(s), or patient activity too soon after surgery. Even though the implant is mechanically fixed (sutured) in position, care is to be taken to assure that the device is rigidly in apposition to the area of the deformity, as demonstrated by total or partial elimination of the visible deformity. If the deformity is not partially eliminated, a secondary Pectus Support Bar may be required as an alternative method of treatment to be considered. During the course of the surgical procedure, and during implantation, extreme care is to be taken to avoid contact with the heart and lungs with either the implant or instruments, as contact to these organs can cause death or permanent injury to the patient. When considering removal, the surgeon should weigh the risks versus benefits when deciding when to remove the implant. When evidence of adequate remodeling is present, removal should be performed. Implant removal is to be followed by post operative monitoring to check for recurrence of the deformity. Where recurrence is encountered, a secondary treatment or alternative treatment may be necessary.

Surgical implants should never be reused. Even though the implant may appear undamaged, it may have imperfections, defects, or internal stress patterns which may lead to breakage or inadequate performance.

#### PATIENT WARNINGS

Post operative care and monitoring is important. Metallic fixation devices cannot withstand activity levels and loads equal to those placed on a normal healthy chest wall. The implant can loosen, migrate, bend, or break as a result of weight bearing, load bearing, strenuous activity, or traumatic injury. The patient is to be warned of the limitations of the device at a hole. The implant can become dislodged, shift, or flip as a result of improper device selection, improper stabilization, not suturing the device(s), or patient activity too soon after surgery. Even though the implant is mechanically fixed (sutured) in position, care is to be taken to assure that the device is rigidly in apposition to the area of the deformity, as demonstrated by total or partial elimination of the visible deformity. If the deformity is not partially eliminated, a secondary Pectus Support Bar may be required as an alternative method of treatment to be considered. During the course of the surgical procedure, and during implantation, extreme care is to be taken to avoid contact with the heart and lungs with either the implant or instruments, as contact to these organs can cause death or permanent injury to the patient. When considering removal, the surgeon should weigh the risks versus benefits when deciding when to remove the implant. When evidence of adequate remodeling is present, removal should be performed. Implant removal is to be followed by post operative monitoring to check for recurrence of the deformity. Where recurrence is encountered, a secondary treatment or alternative treatment may be necessary.

#### POSSIBLE ADVERSE EFFECTS

- Metal sensitivity reactions or allergic reaction to the implant material.
- Pain, discomfort, or abnormal sensation due to the presence of the device.
- Surgical trauma; permanent or temporary nerve damage, permanent or temporary damage to heart, lungs, and other organs, body structures or tissues
- Skin irritation, infection, and pneumothorax.
- Fracture, breakage, migration, or loosening of the implant.
- Inadequate or incomplete remodeling of the deformity or return of deformity, prior to or after removal of implant.
- Permanent injury or death.

#### STERILITY

Steam sterilized the Pectus Support Bar prior to implantation using steam sterilization equipment which has been properly validated. Following the Pectus Support Bar may be required as an alternative method of treatment to be considered. During the course of the surgical procedure, and during implantation, extreme care is to be taken to avoid contact with the heart and lungs with either the implant or instruments, as contact to these organs can cause death or permanent injury to the patient. When considering removal, the surgeon should weigh the risks versus benefits when deciding when to remove the implant. When evidence of adequate remodeling is present, removal should be performed. Implant removal is to be followed by post operative monitoring to check for recurrence of the deformity. Where recurrence is encountered, a secondary treatment or alternative treatment may be necessary.

Pre-vacuumed Steam Sterilization (HI-VAC) Wrapped:

Temperature: 270 ° Fahrenheit (132° Celsius)  
Time: Four (4) minutes  
Drying Time: Thirty (30) minutes MINIMUM

Health care personnel bear the ultimate responsibility for ensuring that any packaging method or material, including a reusable rigid container system, is suitable for use in sterilization processing and sterility maintenance in a particular health care facility. Testing should be conducted in the health care facility to assure that conditions essential to sterilization can be achieved. Since Biomet Microfixation is not familiar with individual hospital handling methods, cleaning methods and bioburden, Biomet Microfixation cannot assume responsibility for sterility even though the guideline is followed.

**Caution: Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.**

**Operating Surgeons and all personnel involved with handling these products are responsible for attaining appropriate education and training within the scope of the activities with which they are involved in the handling and use of this product.**

<b>Authorized Representative in EC:</b>	Biomet UK, Ltd. Waterton Industrial Estates, Bridgend, South Wales CF31 3XA, U.K. Phone: (+44) 1656 655221 Fax: (+44) 1656 645454
<b>European Sales Headquarters:</b>	Biomet Microfixation Europe Toermalijnring 600 3316 LC Dordrecht Netherlands Phone: (+31) 78 629 29 10 Fax: (+31) 78 629 29 12

Health care personnel bear the ultimate responsibility for ensuring that any packaging method or material, including a reusable rigid container system, is suitable for use in sterilization processing and sterility maintenance in a particular health care facility. Testing should be conducted in the health care facility to assure that conditions essential to sterilization can be achieved. Since Biomet Microfixation is not familiar with individual hospital handling methods, cleaning methods and bioburden, Biomet Microfixation cannot assume responsibility for sterility even though the guideline is followed.

**Caution: Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.**

**Operating Surgeons and all personnel involved with handling these products are responsible for attaining appropriate education and training within the scope of the activities with which they are involved in the handling and use of this product.**

<b>Authorized Representative in EC:</b>	Biomet UK, Ltd. Waterton Industrial Estates, Bridgend, South Wales CF31 3XA, U.K. Phone: (+44) 1656 655221 Fax: (+44) 1656 645454
<b>European Sales Headquarters:</b>	Biomet Microfixation Europe Toermalijnring 600 3316 LC Dordrecht Netherlands Phone: (+31) 78 629 29 10 Fax: (+31) 78 629 29 12

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